


1. Place each electrode on patients' skin...

- Shave any chest hair if necessary.
- Clean skin at each electrode site with alcohol pad.
- Snap wires to electrodes following color scheme
- See diagram 1.

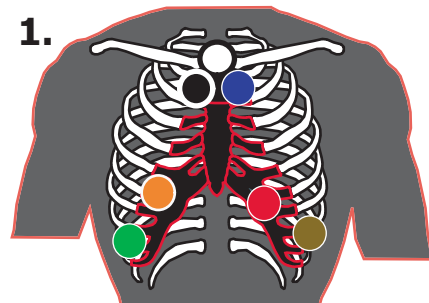

2. Insert "Flash" memory card...

- Remove the cover on the back of the recorder.
- Label Flash card with patients name and date.
- Insert Flash card face up into the recorder.
- Push card into slot until it stops.
- See diagram 2.



3. Insert 2 AA Batteries...

- Insert new batteries; you should hear a "beep".
- Place batteries in correct order.
- Place cover back on the recorder.
- See diagram 3.



4. Wait until "Flash" memory formats...

- Once the Flash card is formatted 100% the Holter will begin recording.
- See diagram 4.



NOTE: Connect electrode wires to recorder; if needed...

- Make sure that the arrow faces forward.
- Once in place, screw down firmly; not too tight.
- See diagram...



**For Hookup Questions
Please call:
1.888.821.4667**